


Do you want to help others?

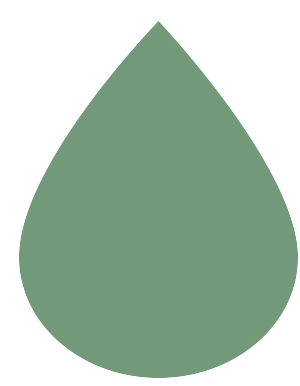

Dear customer.

Thank you for your recent purchase. We hope you love it! If you do, would you consider posting an online review? This helps us to continue providing great products and helps potential buyers to make confident decisions.

Thank you in advance for your review and for being a preferred customer.


<https://www.trustpilot.com/review/hidroxa.com>

Hidroxa's trustpilot



Scan me

Instruction for use of the face electrode

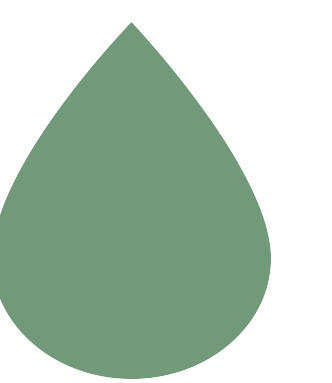

Before putting on the face electrode, soak it in water to the extent that it almost starts dripping. Then, connect the cables to the three electrodes (black pads) and place the electrode onto the face.

It is important to make sure that the face electrode is tightly fastened to the face and that it does not only have contact punctual contact with the skin. If necessary, tighten the strap.

Before turning on the device, connect the cables to the device. For further steps, please see the instruction manual enclosed with the device.

The current for the face treatment should not be set higher than 5 mA. The ideal intensity of current for the face treatment is between 1 to 3 mA (approximately level 0-1.5 for constant current mode or level 2-4 for pulsed current mode).

If it is necessary to set the current higher than 2mA, this must be done under the supervision of a doctor and possible skin irritation are to be carefully monitored.

As a rule of thumb for the optimal amperage, turn up the current until you can feel the current, then turn it back a little so that you can no longer feel the current flow (constant current) or slightly feel it (pulsed current).

INDICATION

If the current cannot be activated, the resistance between the electrodes is too high and the protective circuit against over-therapy is activated. In this case you need to remove the face electrode again. Fill a tub with lukewarm water and add a teaspoon of table salt to it.

Stir the water until the salt is completely dissolved. Soak the face electrode in this tub and apply it to the face once again. Start the treatment as described above.

Please keep in mind - Usually, it takes a few weeks to start feeling the effect of the treatments. If no progress is seen after several weeks, please contact us so we can advise and determine that you did everything you could before ruling out iontophoresis as a possible solution.

For any question you might have, take a look at our FAQ page on our website or contact us anytime via email kundtjanst@hidroxa.com or through our online chat.

Good luck with your treatments!

Read more at our FAQ



Scan me